

Photodon Monitor Hood Installation Instructions

- 1. Gently loosen, but DO NOT remove, the 4 outside thumbscrews.
- 2. Holding the hood in one hand by the top, place the cord over the back of the monitor. Set the hood on top of the monitor with the three rear thumbscrews behind your monitor.
- 3. Adjust the hood by gently sliding each leg towards the middle to touch the side of the monitor.

Hood leg

touching

side of

monitor

centering the middle piece.

Reach behind 4. the monitor to pull the cord up and around the rear center thumbscrew. Reach under the monitor and grab the



cord. Pull it around each of the front bottom corners of the monitor.

5. Center the hood on the top of the monitor with the legs even on both sides. Lightly tighten the thumbscrews.











6. If the hood sides are loose, they may be tightened by pulling the cord through the cord locks on each side.





7. Examine the installation, checking that the hood is sitting securely on the monitor. The three rear thumbscrews should be directly behind the monitor.



8. Install the L-shaped support brackets. Remove the red tape from each bracket to

reveal the adhesive. Place the brackets under the top of the hood, about 4 inches from each corner. Ensuring that the non-adhesive side rests completely against the monitor, press the adhesive side of each bracket firmly to the underside of the hood. Use your other hand as counterforce on the top of the hood.



9. Sitting in front of the monitor, check for light gaps at the top of the monitor. If there are any, place the light-blocking foam to the underside of the hood, directly in front of the monitor. The foam can be trimmed to size before installing.



Find us at: www.photodon.com sales@photodon.com | 847-377-1185 Monday-Friday 9am – 5pm EST